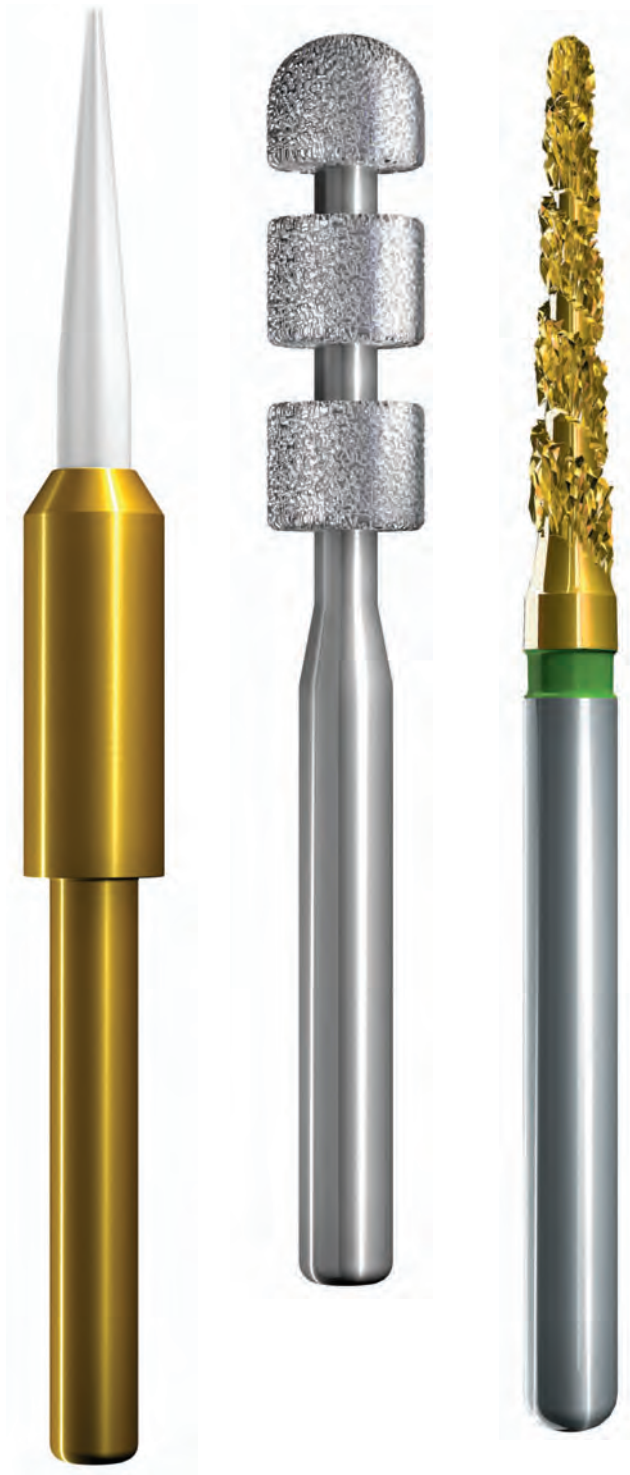
Innovations

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Special shape 370 New grit sizes:

Diamond instrument for contouring the occlusal surfaces of all filling materials.

The three different grit sizes simplify contouring the cusps on both the occlusal and vestibular surfaces.

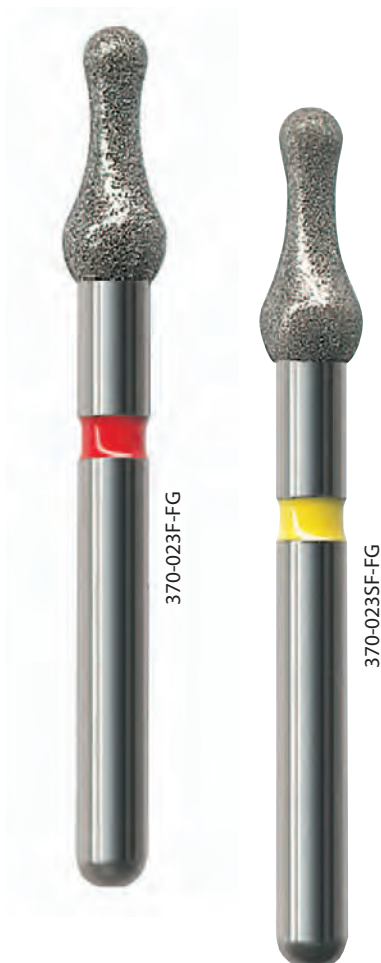
The coarse grit instrument reduces the effort required for pre-contouring.

The fine and superfine grits ready the filling material perfectly for polishing.

The rounded shape of the instrument prevents it cutting grooves in the filling.

The design of this instrument allows it to be used for various applications. The rounded end is for spot grinding accurately.

The rounded concave shape contours occlusal regions perfectly.

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TissueTrimmer, long Longer working section

The successful replacement for scalpels and electrosurgery has been enhanced perfectly.

The new, thinner TissueTrimmer is easier to use in areas which are difficult to access.

The slender tip has the same diameter as the existing TissueTrimmer, which provides for uniform results.

The really narrow, long working section simplifies working in the interdental region

The slender shape simplifies use, especially for deeper incisions.

The long tip can be used for releasing the gingival margin from the tooth, even in deep areas.

This gentle type of incision, excluding necrosis, improves healing by up to 30%.

The minimally invasive incision simplifies regeneration of the gingiva so that even sections which have been removed can grow again.

If used with care, the TissueTrimmer can be re-used up to 300 times – previously unequalled cost-effectiveness!

Perfect for:

- Opening the sulcus for impression taking after preparing the teeth for crown and bridgework.
 - Extending the gingiva for deep class V cavities.
 - Recovering implants after the healing period.
 - Removing granular tissue.
 - Perfect for pacemaker patients.
- Working speed: 300,000 – 500,000 r.p.m.
- **without coolant spray.**

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NTI - VKP Top preparations for all-ceramic restorations

The precise 0.8 mm depth marking meets the recommendations of the manufacturers of all-ceramic systems.

Depth marking

Depth marker 834L:

The edges of the cylinder are rounded to reduce the effort required for finishing the preparation. This enables the shoulder and depth markings to be prepared during the same procedure.

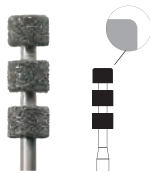
Depth marker 834RL:

This cylindrical depth marker with rounded head prepares perfect shoulders for veneers.

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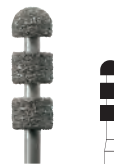
Preparations for all-ceramic restorations NTI - VKP

Diamonds with guide pin

These "nipple diamonds" have been in the NTI range for 10 years and are now available with a preparation depth of 0.8 mm.

The highly precise guide pin prepares a perfect shoulder for all-ceramic restorations.

Available in two basic shapes – cylindrical and conical – depending on which preparation philosophy is preferred.

The length of the working section simplifies preparation, even on longer teeth.

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Order No. 999- ...F -FG 021

998

Diamond Instruments with guide pin

The crown preparation instrument, proven over the past 10 years.

The guide pin makes the preparation of defined shoulders easy, especially in difficult-to-look and difficult-to-reach areas.

A "gutter" preparation will be prevented.

Different sizes allow a selection for each situation.

Two grits:

- super coarse for bulk removal
- fine for perfect surface to allow for an easy crown production by the technician

Both diamonds are cut-to-grit for exact preparations.

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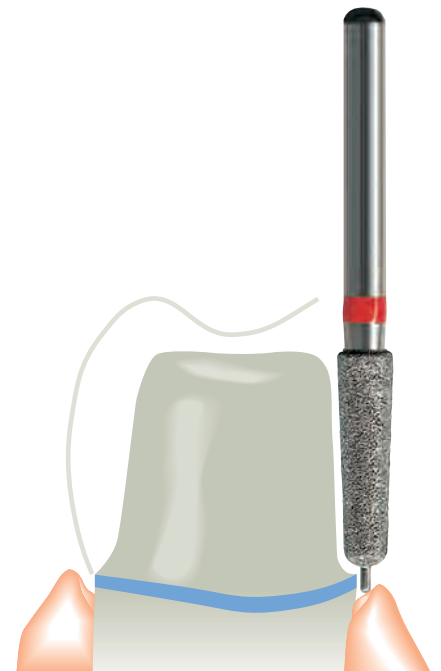
bei 021SC, 023SC = 300.000 min⁻¹

FiberMaster, conical

Glass fibre root posts with conical shape.

The well known FiberMaster posts are now available with conical shape.

The high density of the glass fibres provides the FiberMaster with a modulus of elasticity of 30 GPa* which corresponds to that of dentine. This reduces the risk of the root fracturing.

The FiberMaster is available in three sizes with a head and two sizes without a head. This simplifies selecting the correct post for the specific patient.

Adhering the posts stabilizes the teeth, respectively, roots.

The conical shape of the root section corresponds to the anatomy of a tooth. This reduces the risk of perforating the root while drilling.

There is no bonding agent on the surface of the posts. This allows the dentist to use his composite adhesive of choice.

Posts with heads simplify building up the core and keep the amount of composite to a minimum.

The headless posts are ideal where inadequate space is available.

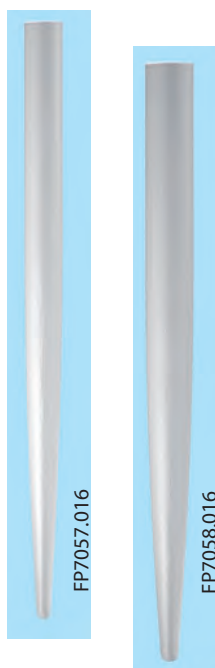
The occlusal sections of the posts can be reduced at any time.

The preparation instruments are identical with those of the RPR post and core system and can be used for titanium posts as well as the new glass fibre posts.

*GPa = Gigapascal

The modulus of elasticity of dentine is approx. 20 GPa

(Literature: Fiber posts - characteristics and clinical applications/ Marco Ferrari and Roberto Scotti by Masson S.p.A- Milan, Italy)

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Assortments: 5 Posts, 1 Root Facer, 1 Root canal drill, 1 Roughening Instrument, 1 Depth gauge

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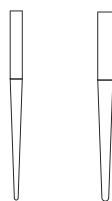
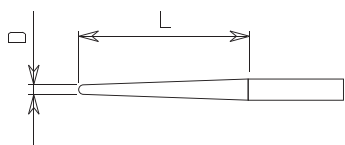
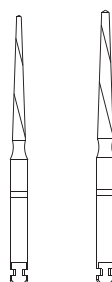
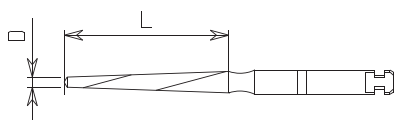
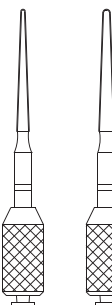
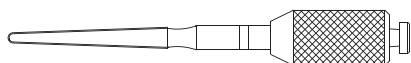
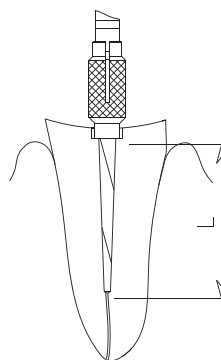
Assortment: 5 Posts, 1 Root canal drill,
1 Roughening Instrument, 1 Depth gauge

Order No. FP7461

Order No. FP7462

H1SX RAL

**H1SX -
Tungsten carbide bur with cross cut**
Excavating with cross cut

Features and advantages:

High cutting capacity blade design for rapid excavation.

Cross cut ensures smooth preparation in the depth of the cavity >>> **very patient friendly.**

Transitional face-end cut smoothes the cavity floor safely and quickly this ensures minimally invasive, atraumatic material reduction.

Slim design of the neck provides a clearer view of preparation area.

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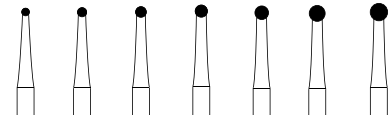
FS3-FX

FS3-SFX

Serrated Diamond Finishing Strips, single sided

for contouring the interdental regions of any type of filling

Removing excess material from the proximal regions of filling

Contouring the proximal regions of all types of filling.

For use on composites, acrylics, glass ionomer cements and amalgam etc..

Removing excess cement after placing inlays, onlays and laminate veneers.

Advantages: The serrated edge separates interdental spaces quickly >>> Simplifies working procedures.

The serrated diamond strips have a wider non-diamond-coated area >>> Simplifies sawing.

Serrated sawing edge and diamond-coated abrasive area combined in one instrument >>> Simplifies working procedures and reduces stock.

UniMatrix® diamond-coating >>> Economical service life.

Contours the proximal regions anatomically. Are sterilizable and re-usable.

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H152-009-FG

Features and advantages:

Special carbide bur for opening and paralleling the cavity walls during endodontic treatment.

The non-cutting tip avoids a via falsa preparation.

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LaboStand

This new bur block for dental laboratories is made of a particularly insensitive material.

This makes it easier and better to clean.

The round design matches the shape of the instruments better.

80 holes can hold your rotary instruments. Its weight of 3 kg prevents the stand from slipping away 20 cm diameter; 13 holes in the back for pre-selected instruments, 12 holes at magnetic strip and 55 holes in the center.

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Plaster Plus

The diamond disk for sectioning ALL models. The new high-strength, top quality disk cuts slender gaps (0.24 mm) even where the teeth are overcrowded.

Segmented according to the latest physical studies to create outstanding stability yet remain see-through while running.

Segmented in the diamond-coated section to allow the dust to escape from the slots. This prevents the disk wobbling and jamming in the cut.

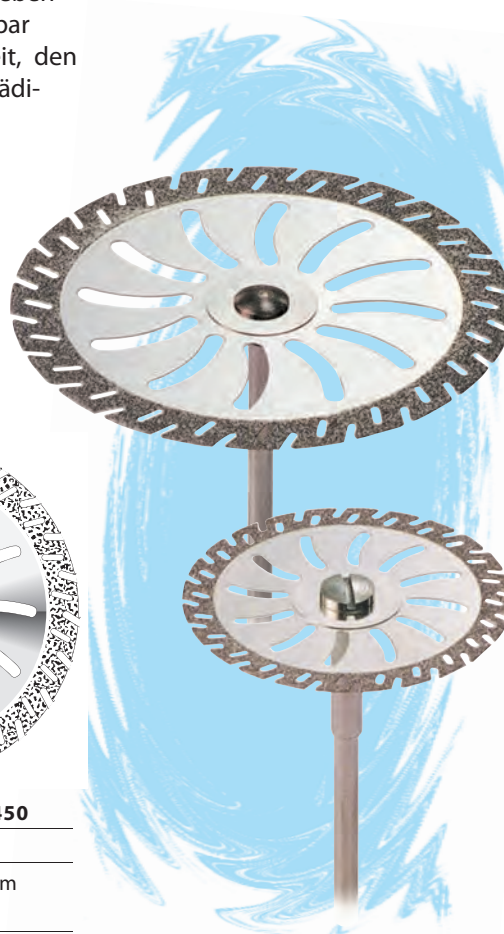
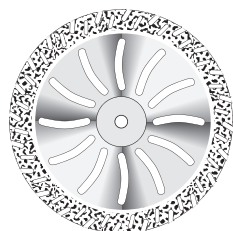
The NTI Unimatrix® diamond-coating is very long lasting.

The 64 µm diamond grit creates a neat cut without splintering the plaster.

As it can also be used on type 2 plasters, it is also suitable for separating the dental arch for orthodontic set-ups.

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Millennium Cutter with a low vibration level

can be used for all dental alloys, porcelain, composites and denture acrylics. However, they cannot be used for titanium.

The special cutting edge ensures smooth grinding and is therefore also suitable for porcelain.

The superfine cutting edge of the cutter creates a smooth grinding surface on the metal which simplifies reworking.

The cutting edge geometry reduces the vibrations in the instrument by 95%. This makes it easier to secure and process even very small objects.

Working with a very low vibration level helps to prevent injury to your wrists and to avoid the development of tenosynovitis and tennis elbow.

Targeted removal can be ensured by varying the speed and the pressure applied.

Available in seven different shapes.

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Milling instruments now available with Millennium coating

Applications: Parallel and 2° conical milling.

Advantages:

The XFR blades are especially coarse for rough-milling all non-precious alloys and titanium.

The XR blades create a smooth surface, which simplifies polishing later.

The special Millennium coating doubles the surface hardness of the tungsten carbide to provide for a long and economical service life.

The blade design and Millennium coating reduce vibration to an absolute minimum during running.

The Millennium coating prevents the chattering and grabbing common with new cutters.

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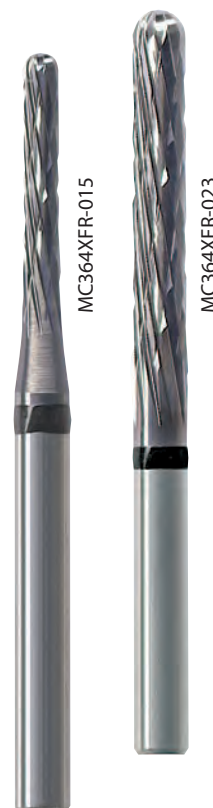
P1269

Cotton buff

High-grade cotton fibres are rivetted firmly to the mandrel. This prevents excessive dust.

The high quality of the base material provides for a long service-life.

The dense bonding simplifies high-lustre polishing with a handpiece.

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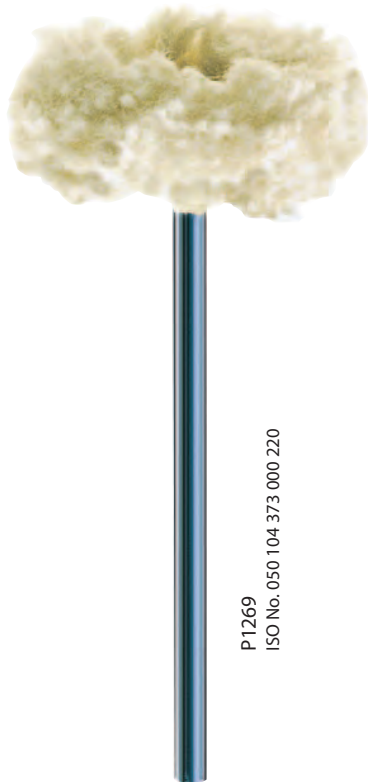
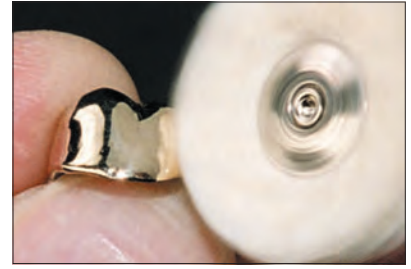
Set-1606

TECH Set by Piet Troost

The Tech Set is developed for the dental office. The useful selection of instruments for little technical work chair side.

The instruments are selected for trimming and polishing the most used technical materials.

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NTI-Kahla GmbH Rotary Dental Instruments