Innovations

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- P13 NTI Unique
 Composite Polierer
Composite Polisher
Polissoirs de composite

Set-1642

Preparation set for CEREC® veneers designed by Dr. Klaus Wiedhahn

Set for preparing CEREC® veneers and final finishing developed in collaboration with Dr Klaus Wiedhahn.

The coordinated instruments for preparing veneers guarantee optimum preparation and consequently accurately fitting restorations.

The coarse grit size of the preparation diamonds produces rapid, time-saving reduction of the tooth structure.

The fine instruments create optimum preparation margins, which are required for accurate recording with an intraoral camera and facilitating digital data processing. This guarantees a gap-free fit of the veneer at the natural tooth.

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Prophylaxe Sets

Prophylaxis Set 1 Set-1643

A complete set for cleaning and polishing teeth, amalgam and composite fillings and precious metal alloys.

The snap-on polishers are used with polishing paste for polishing teeth. The reusable mandrel makes the polishers highly cost-effective.

Abrasive material is integrated in the blue prophylactic polishers. Polishing paste is not therefore required. With very stubborn deposits, polishing paste can be used to increase the polishing capacity.

The small nylon brushes are used with polishing paste. They are ideal for use in the occlusal area and interdental spaces.

The NTI EpsiPol is suitable for use on all precious and semi-precious metal alloys as well as amalgams. The brown prepolisher removes scratches and the green polisher produces a perfect lustre.

The yellow Unique polisher is a diamond-coated one-step polisher for composite fillings. The selected diamond grit allows all types of composite to be polished with a single polisher. This reduces costs and the time required for polishing.

The newly developed silicone binder, which combines flexibility with stability, can be used in all areas and prevents damage to the composite surface.

Prophylaxis Set 2 Set-1644

This prophylactic set is a complete set just for cleaning teeth.

This combination of snap-on polisher, blue prophylactic polisher and small nylon brush facilitates polishing. Different shapes allow use on any type of tooth.

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Instruments with a long shank for crown preparation.

The long shanks allow even large anterior teeth to be prepared palatally without turbine head interference. These instruments facilitate occlusal reduction of tooth structure in the molar region.

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368 (257)

Knospe
 Pointed Football
 Football pointu

Größe/Size/taile ø 1/10 mm	023
Länge - Length in mm - longueur	5,0

Shank/tige		REF	
FGL	806 315 257 524 ...	368- ... M-FGL	023
FGL	806 315 257 514 ...	368- ... F-FGL	023
FGL	806 315 257 504 ...	368- ... SF-FGL	023

	opt.	max.
M	40.000 - 120.000 rpm	300.000 rpm
F	10.000 - 20.000 rpm	40.000 rpm
SF	8.000 - 15.000 rpm	30.000 rpm

379 (277)

Football
 football

Größe/Size/taile ø 1/10 mm	023
Länge - Length in mm - longueur	4,5

Shank/tige		REF	
FGL	806 315 277 524 ...	379- ... M-FGL	023
FGL	806 315 277 514 ...	379- ... F-FGL	023
FGL	806 315 277 504 ...	379- ... SF-FGL	023

	opt.	max.
M	40.000 - 120.000 rpm	300.000 rpm
F	10.000 - 20.000 rpm	40.000 rpm
SF	8.000 - 15.000 rpm	30.000 rpm

879SE-012C-FG

879SE-012F-FG

EC - 2007
Optimum deep chamfers
 - fast - easy - perfect.

The specifically designed shape of the instrument includes a safe-tip for enhancing and simplifying preparation, particularly in the cervical region. It prepares a deep chamfer almost automatically.

Owing to the special design of this instrument, it produces the desired results even in those regions of the preparation which are difficult or even impossible to view. It forms an optimal deep chamfer as ditching is ruled out.

This instrument promotes straightforward yet fast preparation of an optimum deep chamfer as required for fabricating a restoration with excellent precision of fit.

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REF 879SE-012F-FG

879 SE

Torpedo
 Modified Beveled Cylinder
 Torpille

Größe/Size/taille ø 1/10 mm	012
Länge - Length in mm - longueur	10,0

Shank/tige	REF
FG ■	879SE- ... C-FG 012
FG ■	879SE- ... F-FG 012

opt.	max.
C 40.000 - 85.000 rpm	160.000 rpm
F 10.000 - 20.000 rpm	40.000 rpm

846-012M-KID

805-010M-KID

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Kiddies

Microfine diamond instruments with delicate tips for minimally invasive preparation.

The diamond coating with a grit size of 74 µm ensures minimally invasive removal of caries and prevents damage to the healthy tooth structure.

The neck of the shank has a compact design to allow an optimum view of the preparation site, which facilitates treatment.

During prophylactic treatment specific areas of the fissures are opened as required without destroying intact tooth structure.

The round instrument tips produce preparations without sharp edges; no additional preparation is required.

Kiddies with the standard FG shank are manufactured with a shank length of 19.0 mm (ISO 314).

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Kiddies with an FGM shank

In contrast to Kiddies with an FG shank, Kiddies with an FGM shank are manufactured with a shank length 313.

They are ideal for use with children and also adults with a small mouth.

The very delicate instrument tip facilitates cleaning and opening the fissures for sealing.

Kiddies are supplied in FG short shank. Miniature turbines are recommended for best results.

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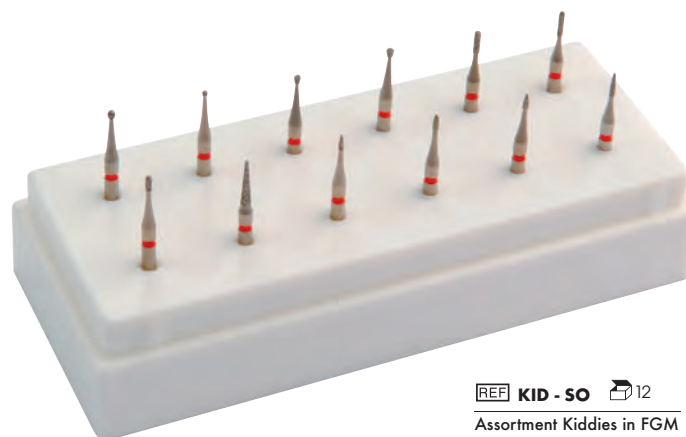
REF KID - SO  12
 Assortment Kiddies in FGM
 Coffret Kiddies en FGM

802L

Round diamond instrument with a diamond-coated neck for opening and expanding the pulp chamber.

The long, diamond-coated instrument neck facilitates preparation of a large opening to provide access for root treatment.

The medium diamond grit size ensures quick, yet minimally invasive preparation.

Two different sizes offer a choice depending on specific requirements.

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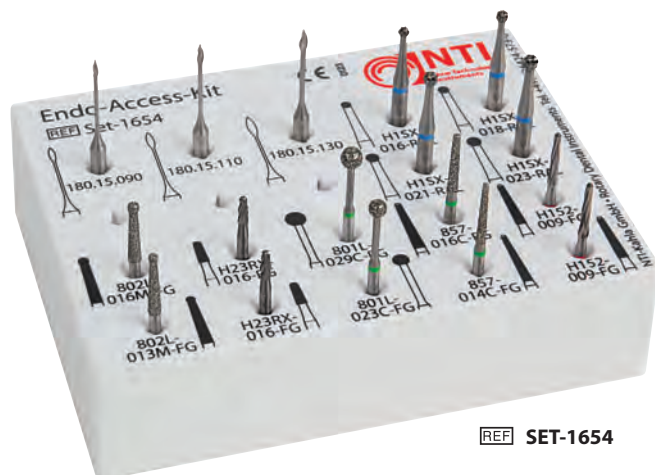
REF SET-1654

Endo-Access-Kit

Well-structured instrument set for quick, safe opening of the pulp chamber prior to endodontic treatment.

The instrument set facilitates precise preparation for endodontic treatment with any indication. The instruments can be used for preparing the natural tooth crown as well as prosthetic crowns.

As the instruments are available in varying sizes, they can be used for any type of tooth.

Instruments with non-cutting ends prevent excessive expansion of root canals and avoid perforation.

Gates drills facilitate preparation in narrow physiological areas.

The tungsten carbide instruments are manufactured in one piece. This reduces the risk of fracture and ensures perfect concentricity.

The special cutting blades of the H23RX-016-FG produce rapid reduction so that preparation of large areas of metal crowns requires less time.

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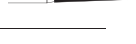
Praxis FG/RA

FG/RA Practice Check Instruments

High-precision shanks for checking correct functioning of dental contra-angles and turbines according to the Medical Devices Directive.

Regularly checked drives ensure precise concentricity, which reduces chattering and vibration and increases patient comfort during treatment.

The check instruments have a smooth surface without a cutting tip so that the dental assistant can use them for checking functioning without any risk of injury.

The instruments have been manufactured to the highest precision, ensuring a precise check of the contra-angles and turbines.

Highly polished shank surfaces prevent damage to the instrument chucks of the turbines and contra-angles.

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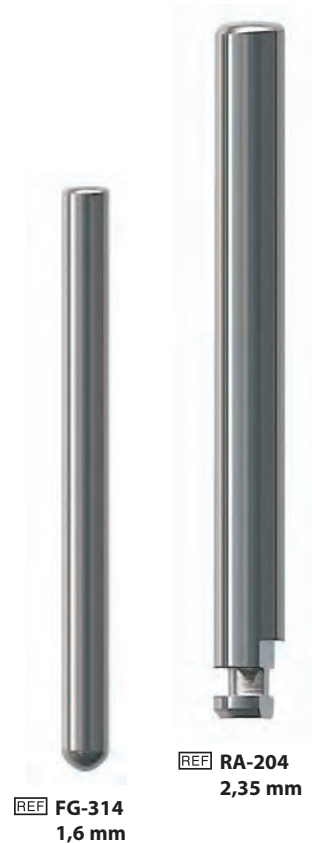
NTI Unique, P20033

NTI Unique, small flame
A really useful new shape for the proven Unique composite polishers.

The small flame shape polishes fissures and cusp slopes quickly.

As this polisher is impregnated with fine diamond grit, it can be used on all types of composite.

*This **single-step polisher** produces a perfect surface very rapidly.*

Exerting higher pressure polishes abrasively to smooth rough zones left after trimming with a diamond.

Exerting light pressure creates a high-lustre surface.

The special coating on the shank simplifies cleaning and sterilisation.

Must be cooled with spray during use!

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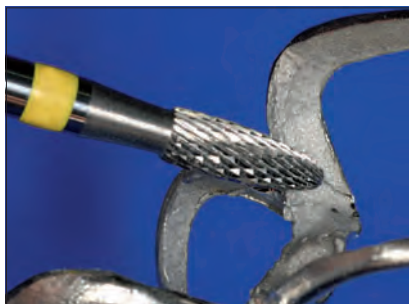
All rights reserved. No part of this catalogue may be reproduced in any form without written permission from the Management of NTI-Kahla GmbH. Reprint, translation, storing and data processing are not permitted without prior approval.

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Products and colours may be subject to alterations. Printing errors excepted.

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Cutter for all types of dental laboratory work

The superfine blades produce smooth surfaces on all types of material and reduce preparation time.

The round shape of the cutting section prevents sharp edges and corners on the surface of the restoration. This avoids damaging or weakening thin areas.

The fine tip can be used for smoothing the inside of clasps and bracing units. Precision is maintained due to minimum removal of material.

The tungsten carbide cutter is also available with the familiar millennium coating. The coating produces a surface hardness of VH 3600 (standard tungsten carbide has VH 1800). The result is a longer service life and outstanding cost-effectiveness.

The superfine cutter blades prevent chattering, so the instrument is very smooth running. This allows it to be used on all types of porcelain facings and increases the application options.

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opt. 5.000 - 40.000 rpm/ max. 40.000 rpm

FS6-M, FS6-F

FS6-M, FS6-F **Diamond strips, wide coated on one side**

Wider diamond strips in two grit sizes for use in filling treatment, in orthodontics and in the dental laboratory.

The six millimetre wide diamond strips facilitate preparing the approximal of fillings and can be used in orthodontics for rapid reduction of teeth that are too wide.

The extra-wide diamond strips can be used for quick, precise preparation of contact points on porcelain bonded crowns.

Two grit sizes are available for rough preparation and smoothing the surface.

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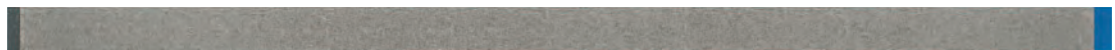
Oliver Brix

Oliver Brix Set

Oliver Brix Set Porcelain set designed by Oliver Brix

For natural contouring the porcelain surfaces of crowns as well as preparing high-performance porcelains and emax®

The set contains a selection of instruments from coarse to superfine for complete surface preparation of porcelain.

If the crown is prepared with the Turbo diamond using an undulating technique, it creates perikymata on the porcelain facing. The shape of the spiral with and without a diamond-coated surface produces an anatomically natural-looking surface in a matter of seconds.

The ultra-thin diamond disc with a non-cutting edge is used to smooth the porcelain surface, preventing scratches and sharp edges.

The CeraGlaze polishers are ideal for finishing an accurately fitting porcelain shoulder.

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PrimeCut SL

The galvanically sintered edge of the diamond disc offers excellent cutting performance

The cutting disc can be used on all alloys and ceramics.

The galvanically sintered edge offers maximum stability and a high diamond density. This guarantees that the cutting disc will have a long service life.

The medium diamond grit size, applied by means of a special manufacturing process, offers faster, gentler material removal. Reducing your work load and preventing poor grinding results.

Stability, in conjunction with the galvanic edge design, means the cutting disc is less prone to breakage. Hence, contouring is considerably simplified.

The cutting wheel is operated at 18,000 – 20,000 r.p.m., using light working pressure (the weight of the handpiece).

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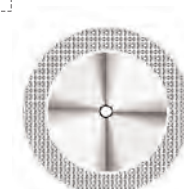
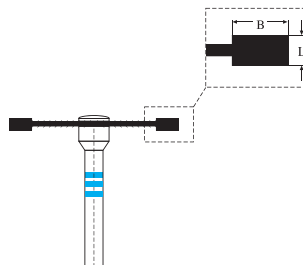
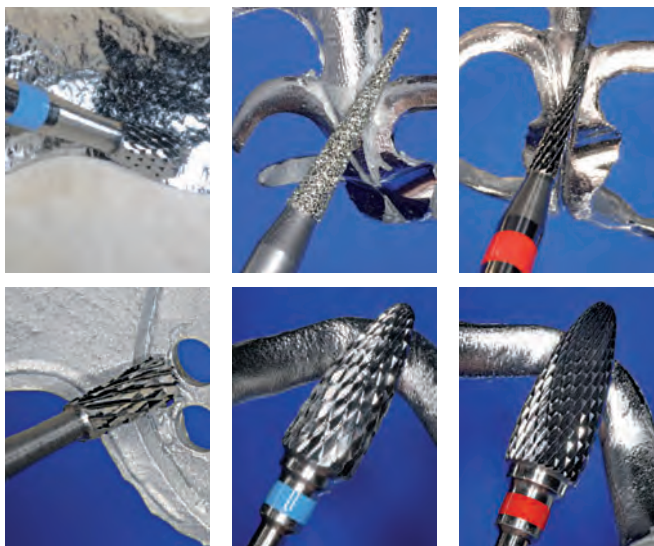
opt. 18.000 - 20.000 rpm/ max. 25.000 rpm

Set-1656

Model cast trimming set according to Klaus Dittmar

This set is for trimming chrome cobalt frameworks efficiently and economically.

The small pointed shapes are for trimming the juncture between clasp stem and clasp as well as fully contouring Bonwill clasps.

The new flame-shape cutter (HF251SFE-023), with its super fine cross-cut, smooths rough surfaces after trimming with coarse instruments.

The special ergonomics of this cutter produce a smooth, non-scored surface on clasps but without impairing their precision.

The flame-shaped sintered diamond smooths all difficult-to-access areas of chrome cobalt frameworks.

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 opt. 200.000 - 300.000 rpm/ max. 300.000 rpm

 C8-FG

Turbo Diamond Disc for use in a laboratory turbine for contouring interdental spaces and cutting through ceramics.

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379L

The shank of the long Football instruments has a very thin neck. This allows the user to view and monitor the area when trimming intracoronally.

The small diameter of the working section permits absolutely precise spot grinding inside crowns and abutments, even in narrow areas.

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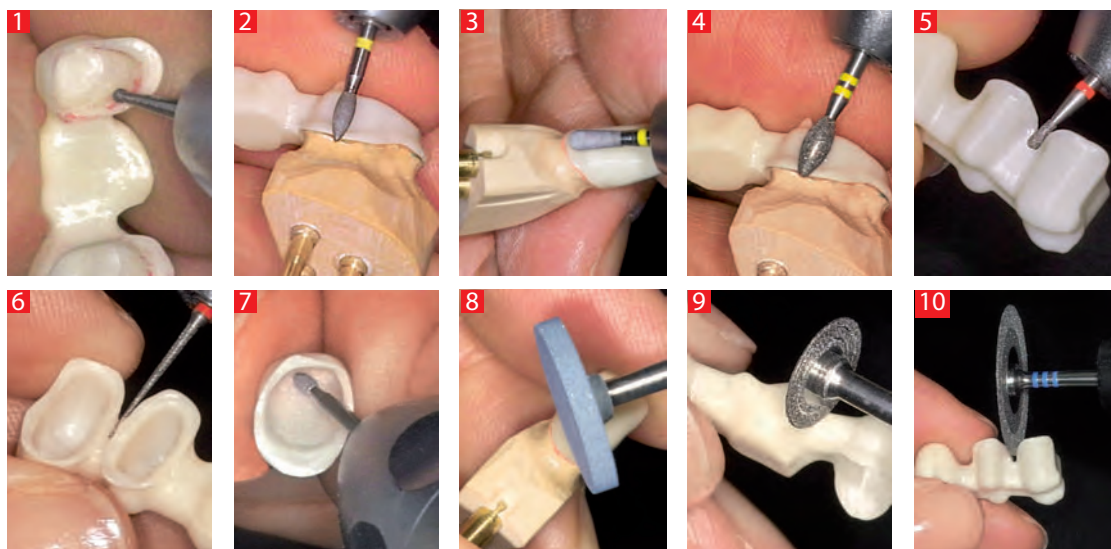
 379L-012F-FGL

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Zirconia set:

Developed together with experienced users, this set includes special instruments for trimming high-performance ceramics innovatively.

CAD/CAM guarantees that these instruments with long-slender necks run perfectly smoothly. This prevents the instrument rattling while trimming the framework material.

The three different grit sizes of CeraGlaze Wheels simplify contouring crown margins accurately. This allows the margin to be reduced on the die.

The grit size of the yellow polisher creates a surface which will not irritate the gingiva when in direct contact with it.

The Turbo Diamond Disc is for widening interdental spaces and contouring using a laboratory turbine.

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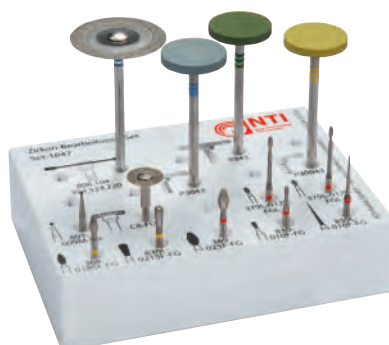
Prüfinstrumente

Labor HP 2,35/3,0 mm

HP 2.35/3.0 mm Laboratory Check Instruments

High-precision instruments for checking the concentricity of handpieces and milling unit handpieces with 2.35 mm and 3 mm chucks.

Check instruments are also available with an FG shank. This now makes it possible to check turbine milling units.

The instruments have been manufactured to the highest precision, ensuring a precise check of handpieces and turbines.

The check instruments have a smooth surface without a cutting tip so that the dental technician can use them for checking correct functioning without any risk of injury.

Checked drives increase the service life of rotary instruments and consequently reduce costs.

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REF HP-124
3,0 mm

G5123

New Sintered Diamond
Flame-shaped sintered diamond with a small diameter for preparing all types of porcelain and alloys.

The round shape prevents sharp or jagged edges on porcelain surfaces.

The 6 mm cutting surface allows anatomical contouring of buccal and occlusal surfaces on porcelain facings and precious metal alloys.

Preparation of the junctions of clasps and bracing arms in CrCo frameworks is reduced to a minimum. The smooth surfaces, produced by applying light pressure and using a motor speed of 15,000 rpm, reduce preparation time.

The selected diamond grit with a stable binder facilitates surface area removal of porcelain and metal and ensures a cost-effective service life.

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e.max® is a registered trademark of Ivoclar-Vivadent AG, Schaan/ Liechtenstein
Cerec® is a registered trademark of Sirona Dental Systems GmbH, Bensheim/Germany