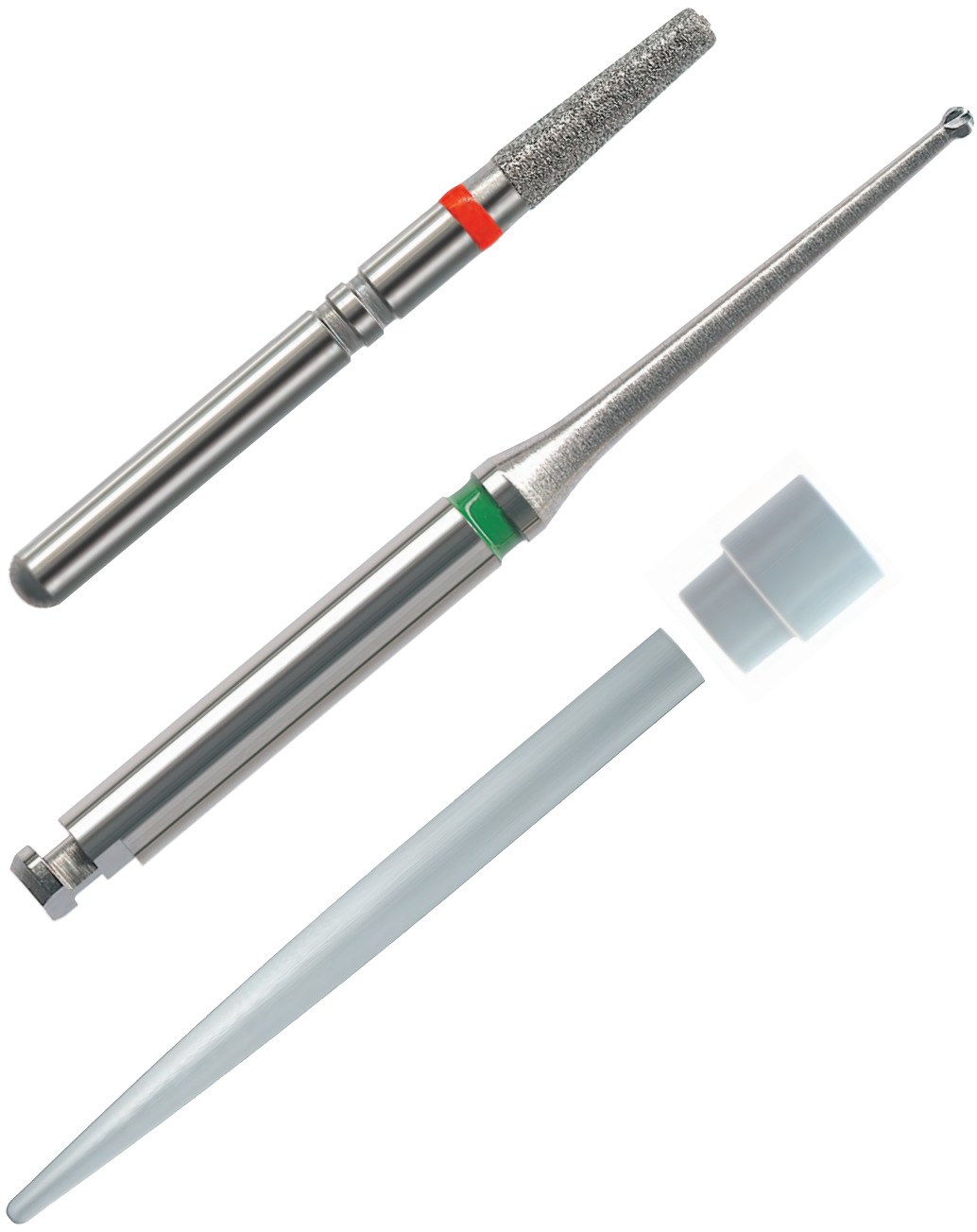
Innovations

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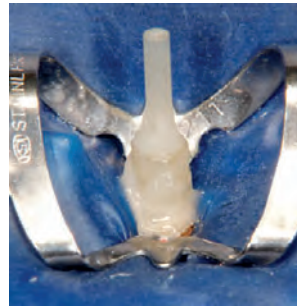
FiberMaster TopHead

A revolution in glass fibre post technology

The tried and tested conical FiberMaster glass fibre posts are amended with a separate head.

For the first time, the head ensures secure fixation of temporary prostheses. Repeat endodontic treatment with simultaneous stabilisation of the tooth is now possible. The head posts feature a two-tier design.

Since this post restoration is inserted two thirds into the prepared tooth, the risk of breakage compared to simple glass fibre posts is significantly reduced.

The heads can be positioned according to the specific length of the posts. This allows the post length in the root to be fully utilised without having to compromise in terms of head stability.

65% of the length of the two-tier head is adhered in the dentin. The axial and lateral masticatory force which occurs is thus completely transferred to the prepared tooth. The risk of breakage, particularly in posts without a head, is almost completely eliminated.

The head can also be used without a post. As such, it is now possible, for the very first time, to securely fix a temporary prosthesis before endodontic treatment has been concluded, even with heavily damaged teeth. The hole in the TopHead enables further endodontic treatment to be performed without losing the retention strength of the temporary prosthesis.

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ABACUS 2000® Diamond Instruments

ABACUS 2000® Diamond Instruments

The tried and tested ABACUS 2000® instruments have now been extended to include an important preparation form and new grit sizes. This further enhances the treatment spectrum. The high density of diamonds and the instruments special coating ensure first-class cost effectiveness.

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Z368 (257)

Knospe
Pointed Football
Football pointu

Größe/Size/taille ø 1/10 mm	020	023
Länge - Length in mm - longueur	5,5	5,0

Shank	ISO	REF	5
FG	Z806 314 257 534...	Z368- ... C- FG	020 023
FG	Z806 314 257 524...	Z368- ... M- FG	020
FG	Z806 314 257 514...	Z368- ... F- FG	020 023

Z845KR (544)

Konus, Kante rund
KR Taper, Modified Shoulder
Cône, bout arrondi

Größe/Size/taille ø 1/10 mm	016	025
Länge - Length in mm - longueur	4,0	4,0

Shank	ISO	REF	5
FG	Z806 314 544 534...	Z845KR- ... C- FG	025
FG	Z806 314 544 524...	Z845KR- ... M- FG	016

Z878 (289)

Torpedo
Beveled Cylinder
Torpille

Größe/Size/taille ø 1/10 mm	010	012	014
Länge - Length in mm - longueur	8,0	8,0	8,0

Shank	ISO	REF	5
FG	Z806 314 289 534...	Z878- ... C- FG	012 014
FG	Z806 314 289 524...	Z878- ... M- FG	010 012 014
FG	Z806 314 289 514...	Z878- ... F- FG	010 012 014

Z879 (290)

Torpedo
Beveled Cylinder
Torpille

Größe/Size/taille ø 1/10 mm	012	014
Länge - Length in mm - longueur	10,0	10,0

Shank	ISO	REF	5
FG	Z806 314 290 534...	Z879- ... C- FG	012 014
FG	Z806 314 290 524...	Z879- ... M- FG	012 014
FG	Z806 314 290 514...	Z879- ... F- FG	012 014

Z881 (141)

Zylinder, rund
Round End Cylinder
Cylindre, bout arrondi

Größe/Size/taille ø 1/10 mm	014	016
Länge - Length in mm - longueur	8,0	8,0

Shank	ISO	REF	5
FG	Z806 314 141 534...	Z881- ... C- FG	014 016
FG	Z806 314 141 524...	Z881- ... M- FG	014
FG	Z806 314 141 514...	Z881- ... F- FG	014 016

Z830L (239)

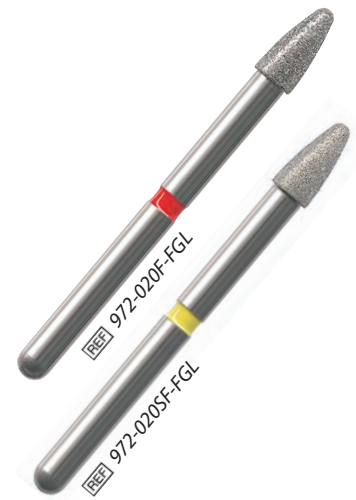
Birne, lang
Long Pear
Poire longue

Größe/Size/taille ø 1/10 mm	012	014
Länge - Length in mm - longueur	4,0	5,0

Shank	ISO	REF	5
FG	TZ806 314 239 534...	Z830L- ... TC- FG	012 014

Okklusal Gestalter

The occlusal shaper for all restoration materials

Shape cusps and fissures with a single instrument.

The round shape facilitates the shaping of cusps and fissures in restorations and also prevents the grinding off of grooves. The fine and super fine grit sizes create an ideal surface for polishing.

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opt. 10.000 - 20.000 rpm, F / 8.000 - 15.000 rpm, SF

Axial Reduction Instruments by Dr Mark J. Caldwell

The new depth markers for veneers

Minimally invasive preparation

Veneers are the minimally invasive solution for aesthetic dentistry. Targeted preparation can assist in removing the minimum amount of material possible and, at the same time, ensuring that a uniform veneer thickness is achieved.

At the courses offered by the "Westerburger Kontakte" dental academy, led by the established expert, Dr Diether Reusch, these depth markers were tried for the first time and recommended for use in aesthetic minimally invasive solutions.

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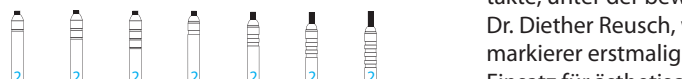
Z-Cut Diamantinstrumente

Z-Cut Diamond Instruments

The abrasive Z-Cut with special grit
 Z-Cut with a grit size of less than 80 µm
 • ideal and rapid results •

Abutments made from zirconia are the perfect solution for all-ceramic superstructures.

These abutments can now be shaped using the new Z-Cut diamond instruments. The NTI special grit size of less than 80 µm prevents the zirconia from being damaged. The formation of cracks and the breaking out of crystals from the zirconia is avoided.

The great stability of the special grit means that the instrument has a very long lifetime. The new bond prevents the diamond grit from breaking out for unbeatable grinding performance.

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opt. 200.000 rpm

K879 (290)

Torpedo
 Modified Beveled Cylinder
 Torpille

Größe/Size/taille ø 1/10 mm	014
Länge - Length in mm - longueur	10,0
Shank ISO  REF	
FG  K806 314 290 524... K879- ...M- FG	014
FG  K806 314 290 514... K879- ...F- FG	014
FG  K806 314 290 504... K879- ...SF- FG	014
FG  K806 314 290 494... K879- ...UF- FG	014

opt. 200.000 rpm

InPrep

InPrep **Minimal effort inlay preparation**

The dental laboratory offers inlays, onlays and half crowns in a wide range of materials. The new CAD/CAM technologies in particular have opened up new options for aesthetic solutions. In the interest of correct functional and instrumental realisation, manufacturers demand smooth cavity floors.

The InPrep achieves exact and super-smooth results in seconds. The instrument features a central inactive guide surface. This prevents unwanted preparation at greater depths.

The precise, specific and necessary preparation depth is reached by means of gentle stroking motions. Thanks to the upright positioning of the InPrep, the inactive guide maintains the set depth and the floor of the cavity is smoothed to produce a smooth finish. Accidental drilling to greater depths is prevented.

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801 in FGXL

The round diamonds now have an extra-long shank and come in a variety of grit sizes

The round diamond facilitates in-depth preparations when it comes to endodontic applications and cavities.

The FGXL shank allows for a better view into the depth of the cavity. The various grit sizes mean processing can be precisely tailored to the patient.

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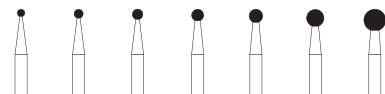
Selective Dentin Remover

Remover for the minimally invasive excavation of caries

The revolutionary cutting geometry guarantees minimally invasive caries excavation.

The newly developed blade configuration which was the product of extensive research, cuts diseased dentin particularly effectively yet cutting performance is reduced when used on healthy dentin.

The angles of the individual blades provide for smooth and vibration-free cutting. This is pleasant for patients and also protects the user's wrist. The instrument's slim neck facilitates the view into the depth of the cavity.

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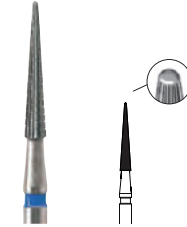
opt. 20.000 rpm

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One-step finishing instruments

Finishing and polishing of composite fillings

With the well-known three-step system with fine, super-fine and ultra-fine tungsten carbide finishing instruments, it takes a certain amount of time and effort in order to subsequently achieve a highly polished finish with the DiaGloss Polishers. The new finishing blade configuration with additional cross cut creates a roughness depth of around 1 μm .

A roughness depth of less than 1 μm is required for pro-hygiene surfaces.

After finishing, the NTI DiaGloss Polishers achieve this surface both reliably and quickly.

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Formen von Titanabutments

Shaping of titanium abutments

Quicker results with two instruments

The instruments marked orange with the aggressive cutting geometry work quickly and effectively and thus save time. The three instruments allow the desired shape to be achieved in all intraoral situations. The round form on the tip prevents the creation of angular preparations in the cervical area and also creates a classic shoulder.

Smooth surfaces are produced both quickly and reliably using the congruent red ring finishing instruments. The instruments' long working parts ensure perfect design results without any rough edges, even in the case of long abutments.

The two-step system reduces the time and effort associated with the processing of abutments in practices and laboratories.

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H152-009-FGL Endo

Tungsten carbide instrument with inactive tip for root canal expansion

The H152 has been on the market for some time now. The new design features a longer shank which facilitates the opening of deep root canals. The inactive tip prevents the creation of a via falsa.

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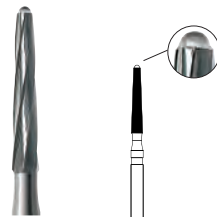
H1SNL Müllerbohrer Endo

Tungsten carbide burs for endodontics

The round Müller carbide burs for the endodontic treatment of root canals.

The small diameters enable the root canals of even the most gracile anterior teeth to be located, opened and extended.

The longer slim neck allows the canals to be reliably located without restricting the view, particularly under the endodontic microscope.

The cutting geometry has been especially designed to make cutting more pleasurable. This makes the removal of even hardened dentin easier.

Three sizes mean you can make just the right selection whatever the situation.

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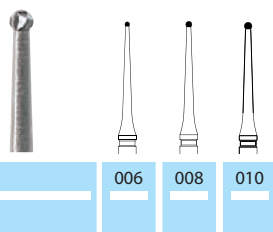
MF-Cut Multifunctional Cutter

H31RMF the pro-cut filling remover

Amalgam removal presents practice teams with a particular challenge. The possibility of mercury vapours is a risk to all involved.

The tungsten carbide bur prevents the amalgam from being ground into tiny pieces which, in turn, reduces the risk of mercury vapours forming.

H31RMF is well-known as an effective crown separator. The pronounced blade configuration cuts large pieces out of the filling and thus minimises the release of mercury vapours. The aggressive blade configuration guarantees that the cavity is opened quickly.

This blade configuration is also suitable for composites. The pro-cut tip eases penetration of and immersion into the filling.

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Thermal disinfection bur block

Bur block for regulation-compliant thermal disinfection of rotary instruments.

Various systems are available for the thermal disinfection of rotating instruments. The tray system in which the instruments are placed without fixation prevents cleaning with water pressure. The tray stops the water jet and organic residue cannot be removed. The instruments are swirled around by the through-flo water. They collide and damage each other.

The second method is a metal frame in which the instruments are clamped. This method can cause damage to the shanks.

The new thermal disinfection bur block combines the advantages of simple application, safe handling and reliable disinfection results in a single device. Two spring steel sheets on the base of the block ensure good fixation in the thermal disinfectors' drawers.

The block is made from high-quality stainless steel. This prevents cross-contamination of oxides with other metals. The silicone holders for the burs mean that the instruments are secured reliably even under the great pressure of the rinsing and cleaning water.

The silicone holders can be exchanged for different sizes at any time. The silicone holders can be individually combined and can thus be used universally for all shank types.

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All rights reserved. No part of this catalogue may be reproduced in any form without written permission from the Management of NTI-Kahla GmbH. Reprint, translation, storing and data processing are not permitted without prior approval.

Products and colours may be subject to alterations. Printing errors excepted.

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© NTI-Kahla GmbH Rotary Dental Instruments • Printed in Germany

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Die Folie

The Film

The removal of premature contacts – quite simply, simple!

Crowns, bridges and prostheses are part of the daily routine in all laboratories. However, the removal of premature contacts takes up a lot of time since the marking film and rotary instrument have to be changed regularly.

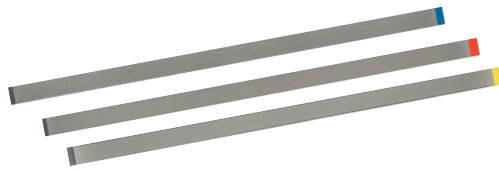
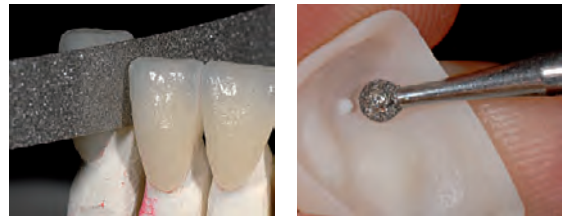
The Film is a highly flexible and ultra-thin instrument which effortlessly handles just this task. The abrasive material which has been specially designed for this task, marks and removes premature contacts in a single step.

The high degree of flexibility allows The Film to optimally adapt to the anatomical conditions. The Film is ultra-thin which means that the occlusion is not blocked during the removal of premature contacts. The gentle movements in the articulator are safely transferred to the premature contact and it is removed.

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Lithium disilicate ceramic preparation set by Oliver Brix

IPS e.max® is made of lithium disilicate glass ceramic. This high-strength ceramic requires selected, durable instruments for proper processing. The veneering material IPS e.max® Ceram produces perfect, natural-looking aesthetics. This is achieved, on the one hand, thanks to careful specific layering to create a vital ceramic and, on the other hand, through formative and structural shaping using rotating instruments.

In order to achieve natural-looking aesthetics, Oliver Brix has not only reworked the set but has also recorded an explanatory video which comes with the set. The video shows how to produce perfect, shade-true, anatomical and aesthetic dental prostheses in a step-by-step guide. From separating the IPS e.max® copings, reworking and layering with IPS e.max Ceram® through to glaze firing, the techniques are demonstrated clearly and simply.

IPS e.max® and IPS e.max Ceram® are registered trademarks of Ivoclar-Vivadent AG, Schaan Liechtenstein.

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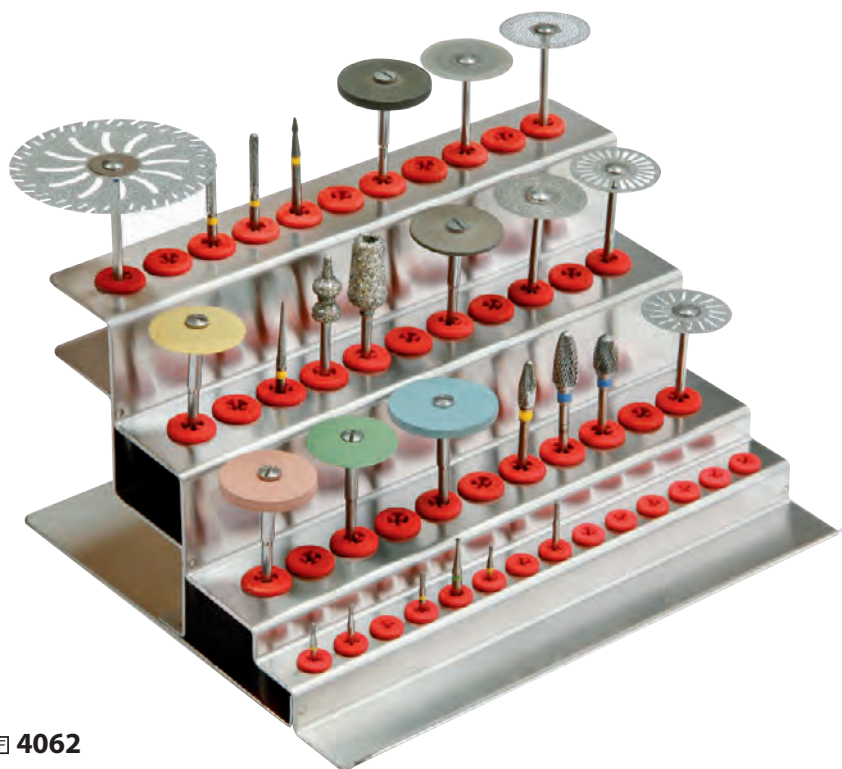
Bohrerständer Labor

Laboratory bur block

This new bur block impresses with its perfect functionality, reliable hold and clarity. The instruments are stored in levels which makes recognising the colour coding even easier.

The diamond discs can be placed next to each other without touching. The spacing between the levels prevents the discs impeding each other when they are removed. The high-quality and sensitive instruments are thus protected from damage.

The bottom row provides space for FG instruments which means that even instruments for zirconia processing can be clearly positioned.

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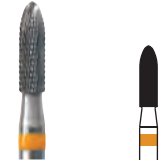
opt. 20.000 rpm

NE - Fräser

Cutters for particularly uniform surfaces on non-precious metals

Modern non-precious metal alloys present special challenges in terms of instruments machining techniques. The newly developed blade configuration of this cutter was designed with perfectly uniform surfaces and cost efficiency in mind.

The instruments optimised cutting geometry ensures the good removal of material. Chipping of the cutting edges is thus avoided for a long service life. The negative cutting angle guarantees a high removal rate and working time is also reduced.

The gentle working pressure produces a previously unobtainable surface which makes polishing easier.

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REF Set-1794

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The Co.Ke set by Jürgen Freitag

The ideal combination of rotating instruments for use on composite and ceramic materials. Jürgen Freitag is an internationally renowned speaker with many years of experience.

The experience he has gained over the years has been applied to this set. The defined goal was to achieve an aesthetic and natural-looking result with just a few instruments.

The selection of instruments allows every step to be performed individually, from shaping to final polishing.

The speeds stated are in line with Jürgen Freitag's unique approach, produce perfect results.

As well as slim-line instruments for shaping fissures and cusps, the set also includes polishing instruments for a high-gloss finish. For the first time ever, two veneering materials can be processed with just 20 instruments.

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